ManukaLind

Wundheilsalbe 🇧🇪

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ManukaLind Wound Healing Ointment 🇬🇧

Wound healing ointment. For application to the skin. Made with Manuka honey (MGO 514+).

ManukaLind is a medical product for wound care and can be used in all stages of wound healing.

Instructions for use
Read all the instructions in this leaflet carefully. It contains important information. Keep the Instructions for Use.

Areas of application / Indications
ManukaLind can be used for skin irritations and wounds. These include: Superficial wounds, skin irritations, cut wounds, abrasions, scalds, burns (1st and 2nd degree), acute and chronic wounds, ulcers, infected wounds, necrotic wounds, operation wounds, surgical wounds, diabetic wounds, covered wounds, decubitus ulcers, poorly healing wounds, bite wounds, scar care (for sutured wounds), foot and leg ulcers (venous, arterial)

The natural triple effect of ManukaLind – Cleansing, protection and wound healing
Thanks to its high osmolality, ManukaLind has an antibacterial effect. This provides effective, natural protection against bacteria and invading germs. Dressing changes are less painful and less traumatizing as the ointment reduces dressing adhesion due to ongoing moisture.

ManukaLind creates an ideal moist and optimum pH wound environment. ManukaLind promotes wound granulation and counteracts wound odour.

Instructions for use

- Can be applied to the wound several times a day as required.
- Carefully clean the wound edges if necessary.
- Apply ManukaLind generously. There must be full contact between ManukaLind and the wound bed.
- A suitable sterile dressing can be used to cover the wound.
- If interactive dressings are used (e.g. polyfoam or hydrocolloids), it must be ensured that they do not contain active ingredients, since no experience has been gained yet regarding the compatibility with ManukaLind.
- The dressing should be changed if it is saturated with exudate, or if the covering dressing leaks wound secretions.
- When removing / changing the dressing, if necessary, rinse / clean the wound with a saline solution and reapply ManukaLind.
- The currently valid guidelines for hygiene and infection prevention must be observed when caring for wounds or changing dressings. This includes at least thorough hand disinfection, the use of sterile disposable gloves and mouth / nose protection.

Contraindications

- Not to be used with known allergy to bee products (e.g. honey, pollen, propolis etc.), thyme or lavender oil.
- Do not use on heavily weeping wounds, third degree burns, in deep tissue cavities / wound areas or sinuses.
- Not suitable for children under 12 months. Pregnant women and breastfeeding mothers after consulting with their doctor.
- There are no reports of elevated blood sugar levels with diabetic patients after use of the ointment, as the product is not resorbed. Nevertheless, diabetics should consult their doctor before using ManukaLind.

Safety Precautions and Warnings

- Do not bring the opening of the tube into direct contact with the wound.
- Use one tube per patient.

- In case of contact with eyes, rinse thoroughly.
- After application, light burning or pulling sensation may occur. In case of ongoing discomfort remove dressing, rinse the wound with a saline solution and if necessary, consult your doctor.
- If allergic reactions occur, stop treatment immediately and contact your doctor.
- If the desired effect is not achieved within 3 weeks, discontinue use and consult your doctor.

Mandatory reporting
Please report all serious incidents related to ManukaLind to the manufacturer and the competent authority in your country.

Ingredients
Macrogol, Manuka honey (MGO 514+), essential oils (lavender oil, thyme oil).

Disposal
Empty packaging can be disposed of with household waste.

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