## ManukaLind

### Wundheilspray

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## ManukaLind Wound Healing Spray

Wound Healing Spray. For application to the skin. Made with Manuka honey (MGO 514+).

ManukaLind is a medical product for wound care and can be used in all stages of wound healing.

#### Instructions for Use

Read all the instructions in this leaflet carefully. It contains important information. Keep the Instructions for Use.

#### Areas of application / Indications

ManukaLind can be used for skin irritations and wounds. These include: Superficial wounds, skin irritations, cut wounds, abrasions, scabs, burns (1st and 2nd degree), acute and chronic wounds, ulcers, infected wounds, surgical wounds, operation wounds, necrotic wounds, diabetic wounds, covered wounds, decubitus ulcers, poorly healing wounds, bite wounds, scar care (for sutured wounds), foot and leg ulcers (venous, arterial)

#### The natural triple effect of ManukaLind – Cleansing, protection and wound healing

Thanks to its high osmolarity, ManukaLind has an antibacterial effect. Thus provides effective, natural protection against bacteria and invading germs. Dressing changes are less painful and less traumatising as the spray reduces dressing adhesion due to ongoing moisture.

ManukaLind creates an ideal moist and optimum pH wound environment.

ManukaLind promotes wound granulation and counteracts wound oedema.

#### Instructions for use

• Can be sprayed several times daily as needed for wound care.

• Carefully clean the wound edges if necessary.

• Spray generously on the wound from a distance of 10 cm. It is important to ensure that there is complete contact between the film and the wound base. Avoid direct contact between the spray head and the wound.

• A suitable sterile dressing can be used to cover the wound.

• If interactive dressings are used (e.g. polyfoam or hydrocolloids), it must be ensured that they do not contain active ingredients, since no experience has been gained yet regarding the compatibility with ManukaLind.

• The dressing should be changed if it is saturated with exudate, or if the covering dressing leaks wound excretion.

• When removing / changing the dressing, if necessary, rinse/clean the wound with a saline solution and reapply ManukaLind.

• The currently valid guidelines for hygiene and infection prevention must be observed when caring for wounds or changing dressings. This includes at least thorough hand disinfection, the use of sterile disposables and mouth/nose protection.

• Do not use in patients with a known allergy to bee products (e.g. honey, pollen, propolis etc.) and lavender oil.

• Do not use on heavily weeping wounds, third degree burns, in deep tight cavities/wound areas or sinuses.

• Not suitable for children under 12 Months. Pregnant women and breast-feeding mothers after consulting with their doctor.

• There are no reports of elevated blood sugar levels in patients with diabetes after use of the spray, as the product is not absorbed. Nevertheless, diabetics should consult with their doctor before using ManukaLind.

#### Safety Precautions and Warnings

- Do not bring the spray head into direct contact with the wound.
- In case of contact with eyes, rinse thoroughly.
- After application, light burning or pulling sensation may occur. In case of ongoing discomfort remove dressing, rinse the wound with a saline solution and if necessary, consult your doctor.
- If allergic reactions occur, stop treatment immediately and contact your doctor.
- If the desired effect is not achieved within 3 weeks, discontinue use and consult your doctor.

#### Mandatory reporting

Please report all serious incidents related to ManukaLind to the manufacturer and the competent authority in your country.

#### Ingredients

Glycerol, Manuka honey (514+), essential oils (lavender oil).

#### Disposal

Empty packaging can be disposed of with household waste.

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